A Guide to Safe Handling of Hazardous Materials
Guide de manipulation des produits dangereux

Ansell

www.ansellcanada.ca

CHEMICAL RESISTANCE GUIDE

PERMEATION

Permeation is a process by which a chemical can pass through a protective film without going through pinholes, pores or other visible openings. Individual molecules of the chemical enter the film, and “squirm” through by passing between the molecules of the glove compound or film. In many cases the permeated material may appear unchanged to the human eye.

Chemical permeation can best be described by comparing it to what happens to the air in a balloon after several hours. Although there are no holes or defects, and the balloon is tightly sealed, the air gradually passes through (permeates) its walls and escapes. This simple example uses gas permeation, but the principle is the same with liquids or chemicals.

DEGRADATION

Degradation is a reduction in one or more physical properties of a glove material due to contact with a chemical. Certain glove materials may become hard, stiff, or brittle, or they may grow softer, weaker, and swell to several times their original size. If a chemical has a significant impact on the physical properties of a glove material, its permeation resistance is quickly impaired. For this reason, glove/chemical combinations rated “Limited Protection” or “Not Recommended” in degradation testing were not tested for permeation resistance. Please note, however, that permeation and degradation do not always correlate.

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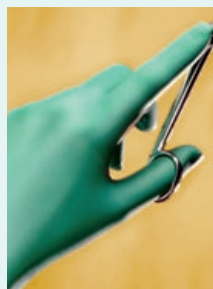
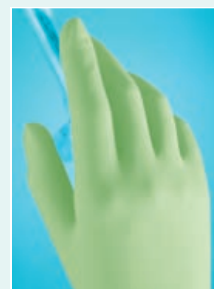
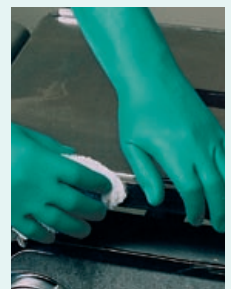
KEY TO DEGRADATION RATING

●	PREFERRED PROTECTION	Little or no degradation or permeation.
●	GOOD PROTECTION	Minimal degradation; permeation should not occur in less than 30 minutes.
●	FAIR PROTECTION	Degradation may occur, and/or some chemical is likely to permeate the glove in less than 30 minutes.
●	LIMITED PROTECTION	Degradation is likely to occur; some chemical will probably permeate or penetrate the glove in less than 5 minutes.
●	NOT RECOMMENDED	Severe degradation and permeation is likely.
○	NOT RATED	Insufficient data is available to make a recommendation.

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Vinyl is not represented in the above chart because Ansell does not recommend for use with chemicals or bodily fluids.

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A

Pure methyl methacrylate (bone cement) rapidly degrades and permeates all types of disposable gloves. If hospital workers must handle cement ingredients that contain high concentrations of methyl methacrylate, Ansell recommends that double gloves are worn and that the outer gloves be replaced very promptly in case of accidental contact. When a bone cement is prepared for use, it is generally prepared in a container using a spatula to mix and avoiding contact with the gloved hand. When mixed the methyl methacrylate begins to polymerize rapidly, thereby increasing in molecular weight and losing the ability to permeate through gloves. When this polymerization has progressed far enough, orthopedic-grade latex gloves, or double gloved latex, neoprene or polyisoprene gloves can be used safely. The time to "progressed far enough" will vary with the temperature, the details of the cement formulation, and other factors. There is no practical way to test for this property. Operating room personnel should therefore be alert for possible skin symptoms. Any medical-grade utility glove may be used for clean-up afterwards. The methyl methacrylate in any unused bone cement will be fully converted to a safe, solid high polymer by then.

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When reviewing this guide remember that barrier ratings are based upon Ansell's knowledge and expertise of materials and their performance in a controlled setting. Actual workplace conditions usually dictate a combination of performance capabilities; these variables cannot be duplicated in a controlled environment. A product's resistance to cuts, punctures, and abrasion must also be taken into account as a critical usage factor. A glove with excellent permeation resistance may not be adequate if it tears or punctures easily. Always factor in the physical performance requirements of the job or application when selecting a glove barrier.

Barrier ratings indicated are intended to guide and inform healthcare personnel, safety specialists and other qualified professionals involved in ensuring safety in the healthcare environment. Because the conditions of ultimate use are beyond our control, and because we cannot run permeation tests in all possible work environments and across all combinations or chemical solutions, this guide is solely advisory. The suitability of the product for a specific job must be determined based upon testing by the purchaser. Ansell believes this information is the best currently available; it is subject to revision as additional knowledge and experience are gained. Anyone intending to use the suggestions contained in this publication should first verify that the glove selected is suitable for the intended use and meets all appropriate health standards.

The suggestions in this guide are not comprehensive and Ansell disclaims any implied warranties, including merchantability and fitness for particular purpose for use of its gloves with any particular chemical or fluid.

Upon written request, Ansell will be happy to provide a sample to aid you in making your own selection to meet your own individual safety requirements.

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© & TM are trademarks owned by Ansell Healthcare Products LLC or one of its affiliates. Printed in Canada. ©2009 All Rights Reserved.

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