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www.ansellcanada.ca

Powder-Free Examination Gloves

In addition to the infection prevention benefits that Ansell exam gloves offer, each glove provides unique features.

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CAUTION: This product contains natural rubber latex which may cause allergic reactions.

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* Does not contain sulphur-based accelerators such as thiurams, carbamates, thiazoles, guanidine or thioureas.

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Proud Partner

Susan G. Komen for the Cure®

Ansell is proud to partner with Susan G. Komen for the Cure® to unite in the fight against breast cancer.

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MICRO-TOUCH® NITRA-TOUCH®

- Nitrile
- Reduces the risk of Type I allergy
- Blue color
- Textured fingertips
- Tested for use with chemotherapy drugs ASTM D6978
- Length: 245 mm (9.6")
- Fingertips thickness (single): 0.16 mm (6.3 mils)
- CGSB certified

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MICRO-TOUCH® NITRILE

- Nitrile
- Reduces the risk of Type I allergy
- Blue color
- Textured finish
- Tested for use with chemotherapy drugs ASTM D6978
- Length: 239 mm (9.4")
- Fingertips thickness (single): 0.12 mm (4.7 mils)

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MICRO-TOUCH® NITRILE E.P.®

- Nitrile
- Reduces the risk of Type I allergy
- Blue color
- Textured fingertips
- Tested for use with chemotherapy drugs ASTM D6978
- Length: 295 mm (11.6")
- Fingertips thickness (single): 0.12 mm (4.7 mils)

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1 Sterile Glove/Package – 100 Gloves/Dispenser Box – 4 Boxes/Case /

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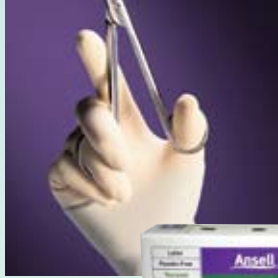
Surgical Gloves

In addition to the barrier protection and infection prevention benefits that Ansell surgical gloves offer, each glove provides unique features.

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These latex gloves contain 50 micrograms or less of total water-extractible protein per gram. *

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Ansell Gloves tested for use with chemotherapy drugs

PRODUCT NAME	PRODUCT CODE	CHEMOTHERAPY DRUGS TESTED																				
		Bleomycin Sulfate	Carboplatin	Carmustine	Cisplatin	Cyclophosphamide	Cytarabine	Dacarbazine	Daunorubicin	Doxorubicin Hydrochloride	Etoposide	5-Fluoro-uracil	Idarubicin	Ifosfamide	Mechloroethamine	Melphalan	Methotrexate	Mitomycin	Mitoxantrone	Paclitaxel	Thiotepa	Vincristine
Latex Powder-Free Examination Gloves																						
Micro-Touch® E.P.® Gloves	920155 / 920785	●	●	●	●	●	●			●	●	●	●	●	●	●	●	●	●	●	●	
Synthetic Powder-Free Examination Gloves																						
Micro-Touch® Affinity® - Neoprene	3770 / 3774		●	●	●					●	●	●			●					●		
Micro-Touch® NitraFree™ - Nitrile	6034510 / 6034514					▲				▲	▲	▲				▲				▲	▲	▲
Micro-Touch® Nitra-Touch® - Nitrile	6034010 / 6034014			▲	▲	▲				▲	▲	▲		▲						▲	▲	
Micro-Touch® Nitrile	4000 / 4008	●	●	●	●	●	●	▲		●	●	●	●	●	●	●		●	●	●	●	●
Micro-Touch® Nitrile E.P.®	6034050 / 6034054			●	●	●				●	●	●	●	▲						▲	▲	●
Procedure Gloves																						
Micro-Touch® NitraTex® Sterile - Nitrile – EACH	6034251 / 6034253			▲		▲					▲	▲	▲							▲	▲	▲
– PAIR	6034151 / 6034153																					
Latex Powder-Free Surgical Gloves																						
Encore®	5785000-1 / 5785007-1					▲				▲	▲	▲				▲				▲	▲	▲
Encore Acclaim®	5795000 / 5795007					▲				▲	▲	▲				▲				▲	▲	▲
Encore® Orthopaedic	5788001 / 5788007					▲				▲	▲	▲				▲				▲	▲	▲
No Powder®	860260 / 860785	●	●	●	●	●	●			●	●	●	●	●	●	●	●	●	●	●	●	●
Synthetic Powder-Free Surgical Gloves																						
Derma Prene® IsoTouch® - Polyisoprene	20685255 / 20685290			▲		▲				▲	▲	▲				▲				▲	▲	▲
Derma Prene® Ultra - Neoprene	851155 / 851890	●	●	●	●	●	●			●	●	●	●	●	●	●	●	●	●	●	●	●

ASTM TESTING: ● F739: Standard test method for resistance of protective clothing materials to permeation by liquids or gases under conditions of continuous contact
▲ D6978: Standard test method for permeation by chemotherapy drugs

Tested for use with chemotherapy drugs

Ansell chemotherapy tested gloves have been tested either by ASTM F 739 or by ASTM D 6978. Specific chemical resistance data are available upon request by contacting our Customer Service Representatives. Gloves used for protection against chemotherapy drugs must be selected specifically for the type of chemicals used. Review material safety data sheets for the chemicals being used and determine the required level of protection.

Safety and Protection

Hazardous Drug Dangers

The hazards encountered during the preparation, administration, clean up and general handling of any cytotoxic drug have been well documented. Public concern is magnified by the perception that cancer risks are imposed upon workers, nurses and pharmacists that are involved in these procedures.

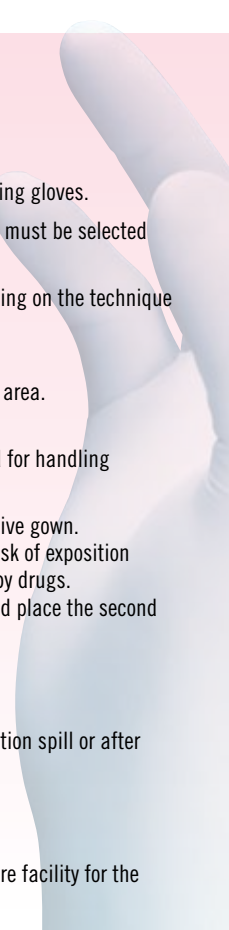
The dangers of handling hazardous drugs come from their inherent toxicity as well as the type of exposure that may occur in the workplace. Today that workplace environment may be a hospital clinic, other patient unit, a physician's office, pharmacy, or in the patient's home. Chemotherapy administration has also evolved over the years. Chemotherapy agents are administered via intravenous push as well as a continuous infusion through a variety of implanted access devices. Exposure, to the healthcare worker or a family member in the home, may be through ingestion of food that may have been exposed to the drug, inhalation of drug particles that may become aerosolized during preparation, handling of patient excreta during home care, or direct skin contact of the drug.

Studies have been conducted to assess the indirect exposure of healthcare workers, i.e. nurses and pharmacists, to determine if there was any chromosomal damage to these workers that prepared and administered these drugs. This and other reports point to the need for excellent exposure control measures and excellent compliance to safe handling procedures.

Recommendations

Similar care should be taken in the preparation, administration and disposal of chemotherapy drugs. In addition to healthcare professionals, patients and persons that are directly involved in home care should also be taught these safety standards and guidelines.

- Always wear gloves when handling the drug
 - Avoid touching the drug
 - Wash hands before putting gloves on and after wearing gloves.
- Gloves used for protection against chemotherapy drugs must be selected specifically for the type of chemicals used.
- Use medical grade, sterile or non sterile, gloves depending on the technique recommended by your healthcare facility.
- Always use powder-free gloves.
 - Glove powders may contaminate chemotherapy work area.
- Choose a latex, nitrile or neoprene glove
 - Polyvinyl chloride (PVC) gloves are not recommended for handling chemotherapy drugs.
- Use thick gloves that extend over the cuff of the protective gown.
 - Double-gloving is strongly advised when there is a risk of exposition during preparation or administration of chemotherapy drugs.
 - If double-gloving, insert first glove under the cuff and place the second glove over the cuff.
- Before handling chemotherapy drugs,
 - Observe gloves for any break in barrier.
- Change gloves after each use, tear, puncture or medication spill or after 30 minutes of wear.
- Upon removal, both gloves need to be turned inside out
 - Never flick, snap, or toss your gloves.
- Follow recommendations implemented by your healthcare facility for the disposal of gloves.

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Why Testing for Permeation by Chemotherapy drugs?

According to the World Health Organization, between now and 2020 the cancer rate is going to increase by 50% to about 15 million new cases a year, worldwide.

The predicted sharp increase in new cases will mainly be due to the existence of steadily ageing populations in both developed and developing countries, but also to the current prevalence of smoking and the rise in numbers of those adopting unhealthy lifestyles.

One of the most frequently used treatments for cancer is chemotherapy. Unfortunately it is well known that cytostatic agents are potentially hazardous for the manipulator (pharmacist, assistant, nurse, surgeon (HIPEC technique) etc.).

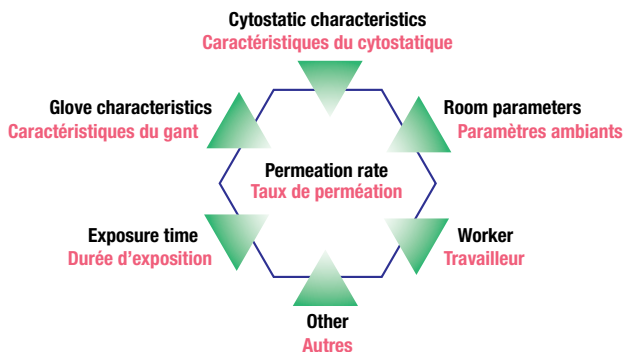
Cytostatics are known to be:

- Mutagenic - induce or increase genetic mutations by causing changes in DNA
- Carcinogenicity in animal models, in patient population, or both,
- Teratogenicity or fertility impairment in animal studies or treated patients.

Some of the other potential effects are :

- Decrease in fertility
- Foetal malformation
- Abortion
- Extra uterine pregnancy
- High toxicity for certain organs.

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Permeation depends on many parameters and therefore the results obtained cannot be extended to any other material, brand or competitive product.

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Ansell Quality Assurance

- Revolutionary anti-slip cuff adheres to all types of gowns
- Eliminates cuff roll-down

CGSB

Canadian General Standards Board delivers independent certification programs where products are assessed against performance and quality standards. Certification programs are a voluntary and impartial mechanism by which suppliers can demonstrate the value of their goods.

Company name and qualified products are displayed on the program list on-line at: pwgsc.gc.ca/cgsb/prgsrv/certprg/program/q020_ind-e.html

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Graham, J.D., Green, L.C., Roberts, M.J., (1988) In Search of Safety « Chemicals and Cancer Risk » Harvard University Press, Cambridge Press

Office of Occupational Medicine, (January 29, 1986) Guidelines for Handling CDs. Occupational Safety and Health Administration Act Publication, Pub 8-11.

Special Feature, (May 1990) ASHP Technical Assistance Bulletin on Handling Cytotoxic and Hazardous Drugs, American Journal of Hospital Pharmacy, 47, 1033-49.

Sotaniemi, E.A., Sutinen, S., Arranto, A.J. et al (1983) Liver Damage in Nurses handling cytostatic agents, Acta Med Scand, 214, 181-9.

Oncology Nursing Society, (1999) Cancer Chemotherapy Guidelines and Recommendations for Practice, 2th edition, 27.

World Health Organization , Global cancer rates could increase by 50% to 15 million by 2020, <http://www.who.int/mediacentre/news/releases/2003/pr27/en/> retrived in March 23, 2009 from.

For additional information and samples, call Ansell Canada at 1 800 363-8340

® & TM are trademarks owned by Ansell Healthcare Products LLC or one of its affiliates. ©2009 All Rights Reserved.

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® & TM sont des marques déposées de Ansell Healthcare Products LLC ou une société affiliée. ©2009 Tous droits réservés.